# freealgin

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**SOLO PER USO DENTALE**

## HYDROPHILIC ADDITION SILICONE (VINYLPOLYSILOXANE) SUITABLE FOR HIGH PRECISION IMPRESSIONS

### CHARACTERISTICS

Freealgin is a material suitable for all the applications of conventional alginate and features high dimensional stability, easy mixing, and practicality in preparation and use. In addition, the impressions can be disinfected safely. Freealgin is also available in the 1:1 version in 50ml cartridges as well as in the automatic mixing system with 5:1 ratio.

### USE

- Preliminary impressions
- Impressions for antagonist models
- Surgery impressions
- Impressions for producing models for bridges and temporary crowns
- Impressions for orthodontics
- Impressions for producing bites and surgical guides
- Impressions for producing models for the construction of trays for whitening solutions
- In all cases in which an impression is needed for modifying or expanding a removable prosthesis

### IMPRESSION TRAY: PREPARATION AND ADHESIVES

All standard and/or special trays can be used, provided they are rigid and non-deformable. The range of **Zhermack's Hi-Tray** impression trays are especially recommended.

The adhesion of the impression material to the spoon is a crucial stage for avoiding errors, especially at the moment of extracting the tray from the mouth. To ensure a strong bond, we recommend using Universal Tray Adhesive (Zhermack).

### FREEALGIN CARTRIDGE VERSION

#### PREPARING THE DEVICE:

Assemble the device as shown in the drawings (1-2).

Warning: Before assembling the tip, ensure that the two components (base and catalyst) flow out evenly by applying a light pressure on the dispenser lever and extruding a small amount of material, which must be removed (3).

Subsequently, insert the mixing tip (4).

### FREEALGIN (Maxi Version 380 ml, 5:1)

#### PREPARING THE DEVICE

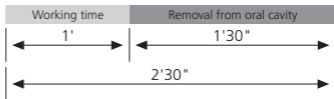
- Press in the levers and remove the protective cap (fig. 5).
- Insert the cartridge into the machine as instructed by the manufacturer.
- Ensure an equal length of material is coming out. Clean and remove any excess material (fig. 6).

- Fit and secure the dynamic-static mixing tip (fig. 7).
- Fit the **mixing tip locker** taking care to engage the clips (fig. 8A) on the cartridge (fig. 8B).
- The Zhermack 5:1 system cartridge is now ready to use (fig. 10).
- Leave the dynamic-static mixing tip in place after use.  
To replace it, simply turn the **mixing tip locker** through 45° and remove it (fig. 11A).  
Then remove the dynamic-static mixing tip (fig. 11B).
- Follow the machine manufacturer's instructions to remove and replace the cartridge.

### WARNINGS

- Always dispense equal amounts of base & catalyst before fitting the dynamic-static mixing tip. If you forget to do so, start the machine and wait until the material coming out is uniform in colour, then take away any non-homogenous material before proceeding.
- Only use original Zhermack 5:1 system cartridges, dynamic-static mixing tips and **mixing tip lockers**. The use of other mixing tips does not guarantee optimal mixing and can jeopardise the product's characteristics and end results.
- The **mixing tip locker** cannot be used with cartridges not from Zhermack's 5:1 mixing system.

### TIMES FOR CLINICAL USE

**WARNING:** In sensitive subjects, vinyl polysiloxanes may cause irritation or other allergic reactions.

### CLEANING AND DISINFECTION OF THE IMPRESSION

After rinsing well under running water, the impression can be immediately disinfected. Ideal disinfecting is achieved by immersing the impression in **Zeta 7 solution** or spraying it with **Zeta 7 spray** (Zhermack). When using other disinfectants, always follow the manufacturer's instructions.

### CASTING MODELS

There is no need to wait; models may be cast immediately after disinfection, or up to two weeks later, provided that the impression is kept at room temperature. Freealgin is compatible with all the best plasters on the market and with polyurethane resins. We recommend Zhermack gypsum, type 4 stone: **Elite Stone** (Zhermack) and type 3: **Elite Model/Elite Model Fast** (Zhermack).

### CLEANING AND DISINFECTING THE DISPENSER

The dispenser can be cleaned, disinfected, and sterilised by immersion in liquid or in a steam autoclave. We recommend **Zeta 3** (Zhermack) disinfectant and **Zeta 2 Sporex** (Zhermack) sterilisation liquid.

## STORAGE

Impressions made with Freealgin must be kept at room temperature. Freealgin impression materials are guaranteed for 3 years following the date of production provided they are kept at a temperature of between 5° and 27°C / 41°- 80°F.

## GALVANISATION

Freealgin impressions may be galvanised with copper or silver.

	freealgin freealgin maxi
ISO 4823 / ADA Spec. 19	Type 2 Medium-Bodied Consistency
Mixing ratio Base : Catalyst	1:1 5:1
Mixing time	Auto mixing System
Total working time*	1'
Time in mouth (minimum)	1' 30''
Setting time*	2' 30''
Strain in compression (min - max)	3 – 5 %
Recovery from deformation	> 99,5 %
Linear dimensional variation after 24h	< 0,20 %
Hardness "Shore A"	40

\*The times mentioned must be intended from the start of mixing at 23°C – 73°F. Higher temperatures reduce the times, lower temperatures increase them.

**IMPORTANT REMARKS:** Consulting services on the use of our products, provided in any form, be that verbal, written or through demonstrations, are based upon the current state of dentistry and of our know-how. They are to be considered as non-binding information, also in relation to possible third party claims, and do not exonerate the user from personally checking whether the product is suitable for the planned use. Use and application by the user take place under no possible control by the Company, and are therefore under the user's responsibility. Possible liability for damages is limited to the value of goods supplied by the Company and utilized by the user.

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