# Zhermack

## gypstray

specific for **trays and instruments**

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#### SOLO PER USO PROFESSIONALE

**EN - INSTRUCTIONS FOR USE:** Gypstray is a ready-to-use solution to remove stone residues from impression trays, brushes and other instruments and aids.

**Packaging:** Gypstray - 3 l container.

**CONTRAINDICATIONS:** No contraindications are known.

**WARNINGS:** May cause organ damage in the event of prolonged or repeated exposure by inhalation. If you feel unwell, consult a doctor. Causes serious eye damage. Causes skin irritation. In the event of contact with eyes or skin, rinse thoroughly for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. If the irritation persists, consult a doctor.

**PRECAUTIONS:** Wear protective gloves and protect the eyes and face. Dispose of the product in compliance with local regulations.

**SIDE EFFECTS:** No side effects have been noted.

**STEP-BY-STEP INSTRUCTIONS:** Pour the quantity of Gypstray required to cover the objects to be cleaned into a container. Eliminate the stone from the instruments or impression trays as far as possible, then immerse in the solution for 15 - 30 minutes. The effervescence indicates the solution is working effectively. Remove the objects from the solution and eliminate any remaining residues. Rinse with water. The power of the solution to eliminate residues can be boosted by using an ultrasonic or thermostatic bath at about 40-50°C.

**IMPORTANT NOTES:** The instructions must be respected, irrespective of possible information given in any other way, including during demonstrations. It is the operator's responsibility to ensure that the product is suitable for the application in question. The Company is not liable for possible damage, including to third parties, deriving from failure to respect the instructions or inappropriate use of the product and in any case within the limits of the value of the products supplied.

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